




The non-slip
button holder




Instructions for use

Designed for easy positioning, safety and fun, Splatz hold Jelly Bean™ and Buddy Button™ switches to keep them still. Splatz are non-slip – so they stay put until you want to move them! Being soft and squishy, they help prevent injury if a child hits their switch too hard. They also protect both the switch and the wire from impact, reducing damage and breakages of the expensive buttons.

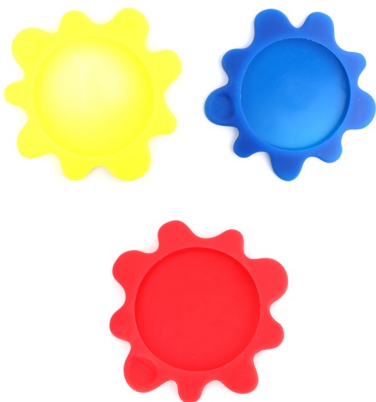

Splatz XL

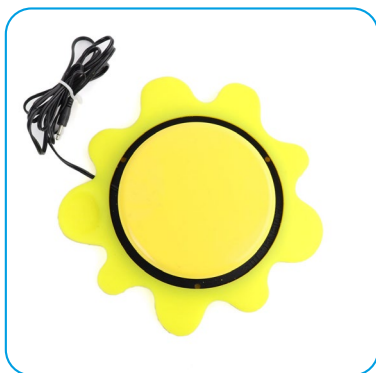

Designed for:
Big Red™ buttons

Splatz



Designed for:
Jelly Bean™
Buddy Button™


How to Set Up

Splatz are very easy to use: Pull the cable through the hole
Push the switch in – if it has a wide base just stretch the Splatz around it.
Pop splatz on your tray or desk and start switching!



How to Clean

Splatz stick best when they're clean so just wash with mild washing up liquid (or put in a dishwasher) and dry thoroughly.

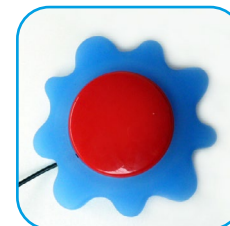

About MERU

MERU is a charity that exists to improve life for children and young people with disabilities. We do this by designing equipment for use at home, at school or college, in hospital or at play. We are here to help meet a child or young person's needs. To help fund our charitable work we manufacture and sell products such as Splatz.

100% of the profits go to MERU!

Warranty

Every Flexzi is warranted by MERU to be free from defects in materials and workmanship for a period of 12 months from the date of delivery. If any defect is found within this period, MERU will repair, or at its discretion, replace the device without charge for materials and labour. This warranty is subject to the provision that the device:

- has been correctly installed
- has been used solely in accordance with these instructions for the purpose for which it was intended
- has not been subjected to misuse or accident, or been modified or repaired by any person other than someone authorised by MERU.

This warranty does not affect your statutory rights.

1 Metcalfe Ave, Carshalton SM5 4AW
+44 (0)1372 725203 | info@meru.org.uk | www.meru.org.uk

Medical Engineering Resource Unit Registered Charity No 269804 Company Limited by Guarantee No 1214125

